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Package Leaflet and Summary of Product Characteristics

Please read carefully

1. NAME OF THE MEDICINAL PRODUCT

Demineralsed human bone matrix, freeze-dried, DIZG

2. COMPOSITION

Demineralsed human bone matrix, freeze-dried, DIZG consists of portions of decalcified human bone tissue.

DBM paste contains: 30% - 32% demineralised bone matrix (DBM) in a mixture of sodium hyaluronate, sodium chloride, sodium monophosphate and sodium diphosphate in water for injection.

DBMx-press contains: 36% demineralised bone matrix (DBM) in a mixture of sodium hyaluronate, sodium chloride, sodium monophosphate and sodium diphosphate in water for injection.

3. PHARMACEUTICAL FORMS:

Chips, strips, granulate, also in paste form.

4. CLINICAL PARTICULARS

4.1 Therapeutic indications:

For implantation. For the treatment of bone defects (e.g. after pseudoarthrosis) as a replacement for lost or insufficient bony structures, for fracture healing disorders, for bone fusion and for performing arthrodeses in the field of oral and maxillofacial surgery and orthopaedics. The type of application and localisation of the demineralised bone matrix to be inserted depends on the given surgical situation and is determined by the attending physician.

The following additional applications are conceivable:

- Reconstruction of meniscus tissue (e.g. after meniscus rupture).

- Operations on the spine in the field of trauma surgery

However, clinical data for these applications are not available.

4.2 Posology and method of administration:

For transplantation according to the clinical indication. The bone defects must be filled loosely as the material swells. The graft must be rehydrated in a suitable physiological medium for at least 30 minutes before use (e.g. in isotonic infusion solution).

Note: Rehydration step is not required for DBM paste and DBMx-press!

4.3 Contraindications:

Use in necrotic host tissue is contraindicated. The indication must be strictly evaluated when applied to poorly vascularised or infected host sites due to a lower rate of integration.

4.4 Special warnings and precautions for use:

For single use only.

4.5 Interactions with other medicinal products and other forms of interaction:

None known.

4.6 Pregnancy and lactation:

There are insufficient data available for use during pregnancy and breast-feeding.

4.7 Effects on ability to drive and use machines:

Not applicable.

4.8 Undesirable effects:

None known.

The following frequencies are applied in the assessment of adverse reactions: Very common (>10%); Common (>1% - <10%); Uncommon (>0.1% - <1%); Rare (>0.01% - <0.1%); Very rare (<0.01% or not known).

Each donor is examined for exclusion criteria based on medical history and physical examination. This is followed by infection serological tests and tests for viral infections using nucleic acid amplification technology (NAT). The results of the laboratory tests required for the procurement of the donor's tissues for: HIV1/2-Ab, HbC-Ab (IgG+IgM) (if positive, then Anti-HBs > 10 IU/l), HbSAg, HCV-Ab, TPHA, HTLV I/II-Ab and HBV-NAT, HCV-NAT, HIV-NAT (PCR) are negative. Despite these extensive examinations, which go beyond the requirements of the EU Directive 2006/17, and the application of a validated chemical cold sterilisation procedure, the transmission of infectious diseases by previously unknown or undetectable pathogens cannot be excluded with absolute certainty.

Reporting of suspected adverse reactions

Reporting suspected adverse reactions after authorisation of the medicinal product is important. It allows continued monitoring of the benefit/risk balance of the medicinal product. Healthcare professionals are asked to report any suspected adverse reactions to the Federal Institute for Vaccines and Biomedicines, Paul-Ehrlich-Institut, Paul-Ehrlich-Straße 51-59, 63225 Langen, Germany, phone: +49 6 10 37 70, fax: +49 61 03 77 12 34, website: www.pei.de. Patients should be informed that they should contact their doctor or healthcare professional if they notice any side effects. This also applies to side effects that are not listed in this Package Leaflet and the Summary of Product Characteristics. Patients can also report side effects directly to the Paul-Ehrlich-Institut. By reporting side effects, patients can help ensure that more information is made available about the safety of this medicine.

4.9 Overdose:

Not applicable.

5. PHARMACOLOGICAL PROPERTIES

Pharmacotherapeutic group:

Graft of human origin.

5.1 Pharmacodynamic and 5.2 Pharmacokinetic properties:

Due to the primarily physical mechanisms of action, there are no studies on pharmacodynamics or pharmacokinetics in patients or trial subjects. Positive experiences with the preparation were documented by the „Study report on the clinical efficacy and tolerability of allogenic avital tissue transplants sterilised with peracetic acid/ethanol“ in 36 patients. This report is based on the statistically analysed experiences of the main hospitals using the product over

a period of four years. It described very good to good results in 100% of all cases in respect of the use of human DBM together with autologous bone tissue and platelet-rich plasma for augmentation in defects of the maxilla or mandible. In addition, reference is made to a placebo-controlled study in which very good defect coverage and very good healing capacity were achieved in 50 patients.

5.3 Preclinical safety data:

Due to decades of use in humans, no preclinical experimental animal studies have been carried out.

6. PHARMACEUTICAL PARTICULARS

6.1 List of excipients:

DBM paste contains: 30% - 32% demineralised bone matrix (DBM) in a mixture of sodium hyaluronate, sodium chloride, sodium monophosphate and sodium diphosphate in water for injection. DBMx-press contains: 36% demineralised bone matrix (DBM) in a mixture of sodium hyaluronate, sodium chloride, sodium monophosphate and sodium diphosphate in water for injection.

6.2 Incompatibilities: None known.

6.3 Shelf life:

5 years in unopened container. DBMx-press 2 years in unopened container.

6.4 Special precautions for storage:

Store from +4°C to +30°C.

6.5 Nature and contents of container:

The content and the outer surface of the inner packaging are sterile.

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