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1. NAME OF THE MEDICINAL PRODUCT

Human cancellous bone, freeze-dried, DIZG

2. COMPOSITION

Spongy section of human bone tissue.

3. PHARMACEUTICAL FORMS

Cubes, chips, granulate, cylinders.

4. CLINICAL DETAILS

4.1 Fields of application:

Intended for implantation.

For the replacement of lost or deficient tissue in the following contexts:

- Oral and maxillofacial surgery: Augmentation of the alveolar ridge in the maxilla, sinus lift, interpositional defects in Le Fort I osteotomy
- Middle ear surgery: Reconstruction of the posterior wall of the ear canal
- Reconstruction of the anterior cruciate ligament: Filling of tibial and femoral tunnels
- Fractures of the radius/tibia: Fractures of the distal radius, depressed fractures of the tibial plateau
- Varus deformity and medial osteoarthritis: High tibial osteotomy
- Ankle surgery: surgical management of pseudoarthrosis
- Spinal surgery: Anterior cervical discectomy and fusion, spinal fusion (spondylodesis)
- Hip replacement surgery: Revision procedures, acetabular reconstruction, total hip replacement

4.2 Dosage, method and duration of administration:

For transplantation according to the clinical indication. The method of administration and exact site depend on the given surgical situation and is determined by the treating physician. The graft must be rehydrated in a suitable physiological medium for at least 30 minutes before use (e.g. in isotonic infusion solution).

4.3 Contraindications:

Use in necrotic host tissue is contraindicated. For host sites that are poorly perfused or infected, the indication must be carefully restricted, as healing is less successful.

4.4 Special warnings and precautions for use:

Designed for single use only.

4.5 Interactions with other medicinal products and other interactions:

None known.

4.6 Pregnancy and breastfeeding:

There is insufficient information available on use during pregnancy or breastfeeding.

4.7 Influence on ability to drive and operate machinery: Not applicable.

4.8 Adverse reactions: None known.

The following frequencies are used to assess adverse reactions: Very common (>10%); Common (>1% – <10%); Uncommon (>0.1% – <1%); Rare (>0.01% – <0.1%); Very rare (<0.01% or not known).

Each donor is examined for exclusion criteria on the basis of medical history and physical examination. This is followed by serological screening for infections, as well as viral testing using nucleic acid amplification techniques (NAT). The results of the laboratory tests required for the procurement of the donor's tissues for: HIV1/2 antibodies, HBC antibodies (IgG+IgM) (if positive, then Anti-HBs > 10 IU/l), HBSAg, HCV antibodies, TPHA, HTLV I/II antibodies, and HBV-NAT, HCV-NAT, HIV-NAT (PCR) are negative. Despite these comprehensive tests, which go beyond the requirements of EU Directive 2006/17, and the use of a validated chemical cold sterilisation method, complete certainty that infectious diseases cannot be transmitted via previously unknown or undetectable pathogens cannot be guaranteed.

Reporting suspected adverse effects

Reporting suspected adverse effects after the authorisation of the medicinal product is important. It ensures the continuous monitoring of the medicinal product's benefit-risk profile. Healthcare professionals are asked to report suspected adverse events to the Federal Institute for Vaccines and Biomedicines, Paul Ehrlich Institute, Paul-Ehrlich-Straße 51 – 59, 63225 Langen, telephone: +49 6 10 37 70, fax: +49 61 03 77 12 34, website: www.pei.de.

Patients should be advised to inform their doctor or healthcare provider if they notice any adverse events. This also applies to adverse events that are not listed in this Package Leaflet and the Summary of Product Characteristics. Patients may also report adverse events directly to the Paul Ehrlich Institute. By reporting adverse events, patients help increase knowledge about the safety of this medicinal product.

4.9 Overdose: Not applicable.

5. PHARMACOLOGICAL PROPERTIES

Pharmacotherapeutic group: Human-derived transplant material.

5.1 Pharmacodynamic and 5.2 Pharmacokinetic properties:

Because the mechanisms of action are largely physical, no pharmacodynamic or pharmacokinetic studies in patients or volunteers are available. Positive outcomes with this product have been documented in 1,205 patients in the "Study report on the clinical efficacy and tolerability of allogeneic non-vital tissue transplants sterilised with peracetic acid/ethanol". This report is based on the statistically evaluated experiences of the main hospitals using the product over a period of four years. It records 90% very good to good outcomes with the use of human cancellous bone as a filler and for covering defects. The healing potential is assessed by

users as very good to good in 82% of cases. Positive outcomes (4 patients) were also noted in bridging defects associated with perioperative fractures after implantation of hip titanium endoprotheses.

5.3 Preclinical safety data:

As the material has been used in humans for decades, no preclinical animal studies have been carried out.

6. PHARMACEUTICAL PARTICULARS

6.1 List of excipients: Not applicable

6.2 Incompatibilities: None known.

6.3 Shelf life: 5 years in unopened container.

6.4 Special precautions for storage:

Store between +4°C and +30°C.

6.5 Container type and contents:

The contents and the outer surface of the inner packaging are sterile.

Cancellous bone cubes/blocks:

1 piece each: 1 cm³, 3 cm³, 9 cm³, 12 cm³, 15 cm³

3 pieces each: 1 cm³, 3 cm³

Cancellous bone chips 1–2 mm: 3 cm³, 5 cm³, 15 cm³, 30 cm³

Cancellous bone chips 2–4 mm: 3 cm³, 5 cm³, 15 cm³, 30 cm³

Cancellous bone chips <1 cm: 5 cm³, 10 cm³, 15 cm³, 20 cm³, 30 cm³

Cortico-spongiosa chips <1 cm: 1 cm³, 3 cm³, 5 cm³, 10 cm³, 15 cm³, 30 cm³, 45 cm³, 60 cm³

Cortico-spongiosa chip, J shape: 1 piece each

10 x 10 – 15 mm, 4 – 5 mm thick; 15 x 20 mm, 10 mm thick

Cortico-spongiosa wedge: Length 25–30 mm, height and width 15 mm each

Granulate:

Particle size 0.212 – 0.85 mm: 0.5 cm³, 1 cm³, 3 cm³, 5 cm³

Particle size 0.25–1 mm: 0.5 cm³, 1.0 cm³, 1.5 cm³

Particle size 1–2 mm: 0.5 cm³, 1.0 cm³, 1.5 cm³, 3 cm³, 5 cm³

Cancellous bone cylinder: 1 piece each

10 x 30 mm (height), 12 x 30 mm (height), 15 x 15 mm (height), 20 x 15 mm (height)

Femoral head:

Endoprosthesis, halved, without cartilage: 1 piece and 2 pieces

Endoprosthesis, quartered, without cartilage: 1 piece and 2 pieces

Halved, without cartilage: 1 piece and 2 pieces

Quartered, without cartilage: 1 piece

Wedge: 1 piece each, 22.5°, 45°

Os ilium, bicortical: 1 piece each

10 – 19 x 40 mm, 10 – 19 x 60 mm, 20 x 20 mm, 20 x 30 mm, 20 x 40 mm, 20 x 60 mm

Os ilium, tricortical: 1 piece each

10 – 19 x 40 mm, 10 – 19 x 60 mm, 20 x 20 mm, 20 x 30 mm, 20 x 40 mm, 20 x 60 mm, 30 x 20 mm, 40 x 20 mm, 60 x 20 mm

Not all pack sizes may be marketed.

6.6 Special precautions for disposal and other handling instructions:

The contents of one pack are intended for use in a single patient only. Any remaining tissue residues must be discarded (disposal according to AVV waste code AS 180102). Once the container has been opened, the graft must be used within 2 hours.

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Email: dizg@dizg.de; Website: www.dizg.de

8. MARKETING AUTHORISATION NUMBER: PEI.H.03355.01

9. DATE OF FIRST AUTHORISATION / RENEWAL OF THE AUTHORISATION: 23 December 2005 / 20 July 2010

10. DATE OF REVISION OF THE TEXT: 28.11.2025

11. GENERAL CLASSIFICATION FOR SUPPLY: Prescription only

Note:

Please use the form on the DIZG homepage to report serious incidents or serious adverse reactions in accordance with Sections 8 and 9 TPG-GewV (Transplant Act-Tissue Ordinance).

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